Plasma-SeqSensei™

IVD Software

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Maggio 2022

PSSSWIFU.R1

it

Sommaro

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
The screenshot shows the 'Plasma-SeqSensei™ IVD Software' interface. At the top, there are three tabs: 'RUN PLANNING' (highlighted in blue), 'DATA ANALYSIS', and 'REPORTING'. The 'sysmex' logo is in the top right corner. Below the tabs, the 'Run Settings' section is visible. It includes a 'Sequencing Device' dropdown menu set to 'Illumina NextSeq™ 500/550', a 'Sequencing Kit' dropdown menu set to 'Mid Output Kit v2.5 (99-cycle)', and a 'Plasma-SeqSensei™ Assay' dropdown menu set to 'Breast Cancer'. Below these, there are two rows for plate selection. The first row is for 'Plate A' with a checkbox, 'Article Number' 'ZR15xxxx', and 'Lot Number' 'D2xxxx'. The second row is for 'Plate B' with a checkbox, 'Article Number' 'ZR15xxxx', and 'Lot Number' 'D2xxxx'. At the bottom, there are 'Reset', 'Back', and 'Next' buttons, along with a close button (X) and a help button (?).

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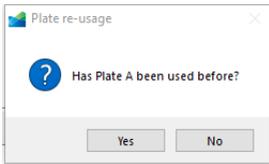Plate Re-Usage Configuration

Plate A

Select sample positions that were used in a previous run by clicking to block them for the automated plate layout configuration. Please note that control positions (column 1 and 12) are blocked by default if a sample position is blocked in the same row.

Layout	1	2	3	4	5	6	7	8	9	10	11	12
A	■	■	■	■	■	■	■	■	■	■	■	■
B	■	■	■	■	■	■	■	■	■	■	■	■
C												
D												
E												
F												
G												
H												

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Sample	Plate	Sample Name	DNA Amount (ng)		Sample	Plate	Sample Name	DNA Amount (ng)	
1	Plate A	Enter sample name...	DNA amount...	X	17	Plate B	Enter sample name...	DNA amount...	X
2	Plate A	Enter sample name...	DNA amount...	X	18	Plate B	Enter sample name...	DNA amount...	X
3	Plate A	Enter sample name...	DNA amount...	X	19	Plate B	Enter sample name...	DNA amount...	X
4	Plate A	Enter sample name...	DNA amount...	X	20	Plate B	Enter sample name...	DNA amount...	X
5	Plate A	Sample1	12	X	21	Plate B	Enter sample name...	DNA amount...	X
6	Plate A	Sample2	4.5	X	22	Plate B	Enter sample name...	DNA amount...	X
7	Plate A	Sample3	15	X	23	Plate B	Enter sample name...	DNA amount...	X
8	Plate A	Sample4	81	X	24	Plate B	Enter sample name...	DNA amount...	X
9	Plate A	Sample5	27.7	X	25	Plate B	Enter sample name...	DNA amount...	X
10	Plate A	Sample6	43	X	26	Plate B	Enter sample name...	DNA amount...	X
11	Plate A	Sample7	90	X	27	Plate B	Enter sample name...	DNA amount...	X
12	Plate A	Enter sample name...	DNA amount...	X	28	Plate B	Enter sample name...	DNA amount...	X
13	Plate A	Enter sample name...	DNA amount...	X	29	Plate B	Enter sample name...	DNA amount...	X
14	Plate A	Enter sample name...	DNA amount...	X	30	Plate B	Enter sample name...	DNA amount...	X
15	Plate A	Enter sample name...	DNA amount...	X	31	Plate B	Enter sample name...	DNA amount...	X
16	Plate A	Enter sample name...	DNA amount...	X	32	Plate B	Enter sample name...	DNA amount...	X

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Plasma-SeqSensei™ IVD Software
 RUN PLANNING DATA ANALYSIS REPORTING

Plate A Summary
 Plasma-SeqSensei™ Assay: Breast Cancer
 Lot Number: D20001
 Article Number: ZR150544


Layout	1	2	3	4	5	6	7	8	9	10	11	12
A												
B												
C	PC											NTC
D		Sample1										Sample2
E		Sample3										Sample4
F												
G												
H												

Plate Locations	Plasma-SeqSensei™ Assay	Sample Name	DNA Amount [ng]	Target Read Count
C09	Breast Cancer	PC	4.3	1202914
C12	Breast Cancer	NTC	0.0	0
D02, D03, D04, D05, D06	Breast Cancer	Sample1	12.0	3490503
D07, D08, D09, C10, C11	Breast Cancer	Sample2	45.0	13111688
D02, D03, D04, D05, D06	Breast Cancer	Sample3	86.0	25058275
D07, D08, D09, D10, D11	Breast Cancer	Sample4	5.0	1456876

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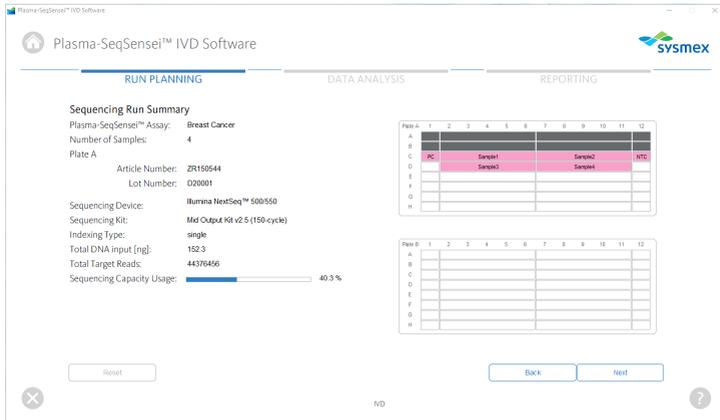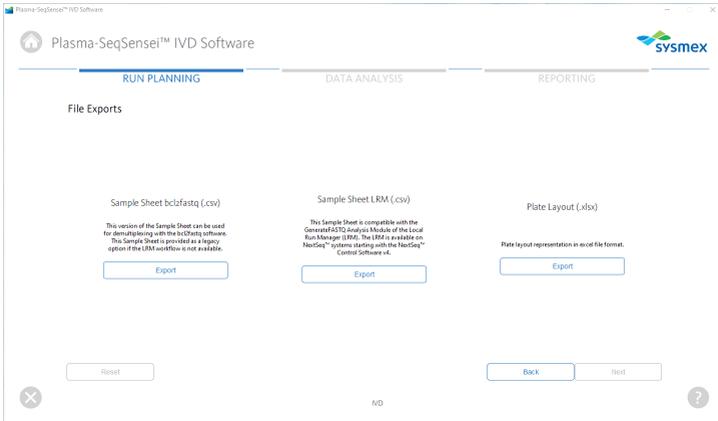
	A	B	C	D	E	F	G	H	I	J	K	L	M
1	Sample												
2	A												
3	B												
4	C	Breast Cancer_JTC	Breast Cancer_Jampt12	Breast Cancer_Jampt13	Breast Cancer_Jampt14	Breast Cancer_Jampt15	Breast Cancer_Jampt16	Breast Cancer_Jampt17	Breast Cancer_Jampt18	Breast Cancer_Jampt19	Breast Cancer_Jampt20	Breast Cancer_Jampt21	Breast Cancer_JTC
5	D	Breast Cancer_Jampt12	Breast Cancer_Jampt13	Breast Cancer_Jampt14	Breast Cancer_Jampt15	Breast Cancer_Jampt16	Breast Cancer_Jampt17	Breast Cancer_Jampt18	Breast Cancer_Jampt19	Breast Cancer_Jampt20	Breast Cancer_Jampt21	Breast Cancer_Jampt22	Breast Cancer_Jampt23
6	E												
7	F												
8	G												
9	H												
10	I												
11	J												
12	Sample	1	2	3	4	5	6	7	8	9	10	11	12
13	A												
14	B												
15	C												
16	D												
17	E												
18	F												
19	G												
20	H												

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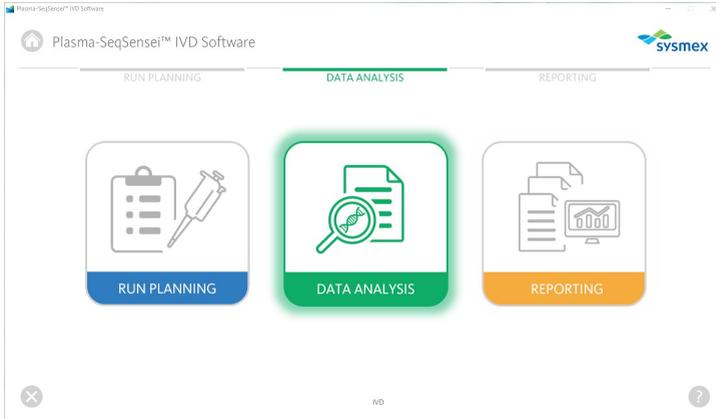
The screenshot shows the 'DATA ANALYSIS' tab of the Plasma-SeqSensei™ IVD Software. The interface is divided into three main sections: 'Analysis Name', 'Sequencing Device' and 'Sequencing Kit', and 'Sequencing Quality Metrics'. The 'Analysis Name' section has a text input field. The 'FASTQ Folder' section has a text input field and a 'Browse' button. The 'Sample Sheet' section has a text input field and a 'Browse' button. The 'Info' section has a large empty text area. The 'Sequencing Device' is set to 'Illumina NextSeq™ 500/550' and the 'Sequencing Kit' is 'Mid Output Kit v2.5 (50 cycles)'. The 'Sequencing Quality Metrics' section includes fields for 'Flowcell ID', '% > Q30', 'Cluster Density [k/mm²]', and 'Clusters Passing Filter [%]'. A 'Start Analysis' button is located at the bottom right of the form.

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*Genome Equivalents: number of amplifiable haploid genomic copies analysed by the assay. 1 GE = 3.3 pg of DNA.

Esempio B:

Sample Information

Sample ID	Analysis Date	DNA Input Amount*	Flowcell ID	Analysis ID	Kit Lot Number	REF Number
	2022-02-03	Not quantifiable	HNLGFAX2			

*Genome Equivalents: number of amplifiable haploid genomic copies analysed by the assay. 1 GE = 3.3 pg of DNA.

The DNA input is listed as not quantifiable if the detected DNA amount for this sample is outside the valid input range. Alternatively, this error message is also displayed if the quantification of the positive control fails regardless of the sample DNA input.

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Run And Sample Validity

Run Validity		Sample Validity	
Positive control:	✓	No template control:	✓
		Sequencing depth:	✓
		Quantification:	✗

For detailed information about the sample validity parameters, please refer to the sample validity section of this report.
Mutation calls will only be reported as valid, if all controls and sample performance criteria are met.
If the detected sample DNA amount is higher than the upper assay limit, mutation calls will be reported as valid but without mutant molecules count.

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Esempio B:

Somatic Mutations Detected

No valid somatic mutations were detected in this sample.

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Esempio A:

Invalid Somatic Mutations Detected

Reason	Number of Invalid Mutations
Failure of Positive Control (PC)	14

Presence of invalid mutation calls detected, please refer to the VCF output for additional information and consider re-running the sample.

Esempio B:

Invalid Somatic Mutations Detected

Reason	Number of Invalid Mutations
Failure of No Template Control (NTC)	14

Presence of invalid mutation calls detected, please refer to the VCF output for additional information and consider re-running the sample.

Esempio C:

Invalid Somatic Mutations Detected

Reason	Number of Invalid Mutations
Sequencing depth of amplicon below threshold	20

Presence of invalid mutation calls, please refer to the VCF output for additional information and consider re-running the sample.

Esempio D:

Invalid Somatic Mutations Detected

Reason	Number of Invalid Mutations
Lower limit for DNA input not reached	17

Presence of invalid mutation calls detected, please refer to the VCF output for additional information and consider re-running the sample.

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Esempio A:

Potential Germline Mutations Detected

Gene ID	Transcript	Coding DNA Change	Amino Acid Change	dbSNP	MAF	Mutant Molecules	Validity
PIK3CA	ENST00000263967	c.363C>T	p.I121=	rs115746478	99.0%	4950	✓
TP53	ENST00000269305	c.329G>T	p.R110L	rs11540654	51.0%	2550	✓

Potential germline mutations were detected in this sample. This classification is based on a mutant allele fraction above 40% and below or equal 60% (heterozygous) or above 90% (homozygous) for the listed mutations.

Esempio B:

Potential Germline Mutations Detected

No potential germline mutations were detected in this sample.

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Sequencing Run Quality

Category	Acceptance Range	Value
Sequencer ID	- / -	
Sequencing Kit	- / -	Mid Output Kit v2.5 (150 cycles)
Clusters Passing Filter	≥ 80%	80.0 %
Cluster Density [Clusters / mm ²]	NextSeq™ 500/550: ≤ 220k	200.0
Percentage >Q30	≥ 80%	80.0 %

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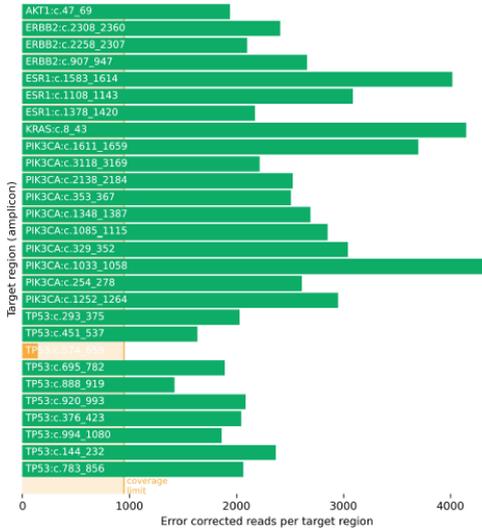
Sample Validity

Sequencing Depth

Sequencing depth per target region: green bars indicate sufficient coverage while orange bars indicate insufficient target coverage.

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Positive Control

	Limits	Result	
Depth of coverage	≥ 1.0 UID families / GE	0.1 UID families / GE (average)	✗
Quantification	$500 \text{ GE} \leq x \text{ GE} \leq 1500 \text{ GE}$	773 GE	✓
Spike-In detection	$\geq 10.0 \text{ MM}$	58.1 MM (average)	✓

If depth of coverage failed, not all amplicons were detected above or equal to the lower coverage limit. For simplicity, only the avg. detection result of all amplicons is shown in this report.

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Esempio A:

No Template Control

NTC ID	Avg. UID Threshold	Avg. UID Result	
NTCplatea	< 15.0	0.03	✓

Esempio B:

No Template Control

NTC ID	Avg. UID Threshold	Avg. UID Result	
NTCplatea	< 15.0	3577.34	✗

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Sample Metrics

Read Overview


Category	Result
Reads imported from sequencer	2,759,170
Reads with sufficient quality and length for error correction	1,831,255
Rate of reads usable for error correction	66.37%
Reads assigned to UID families with sufficient size	1,779,693
Rate of reads assigned to UID families with sufficient size	97.18%
Error corrected reads	101,635

FASTQ reads are imported from the sequencing device. Only reads passing certain quality and length thresholds will be further analysed by the Plasma-SeqSense™ IVD software. Reads are grouped according to their bar-code sequence (UID / UMI) and only reads assigned to UID families with sufficient size are used for error correction. The error corrected reads are the starting point for alignment and mutation calling.

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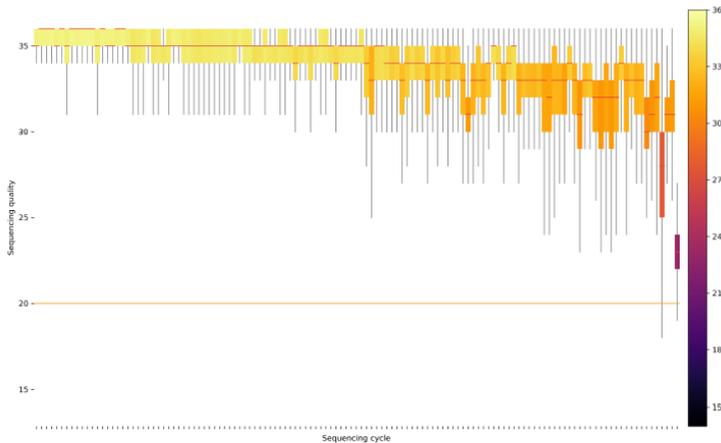Sequencing Quality

Distribution of QPhred quality values in error corrected reads per sequencing cycle. The orange line in the plot indicates the QPhred quality threshold which is required for mutation calling. A subset of 10,000 reads is used per FASTQ file of the sample to generate this plot.

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Analysis Software and Database Information

Analysed by Plasma-SeqSense™ IVD Software

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9 Appendice A

General Terms and Conditions

for Software Licenses of Sysmex Inostics GmbH

1. Subject matter

Sysmex Inostics GmbH, Falkenried 88, 20251 Hamburg, Germany (hereinafter referred to as "SIG") grants the customer a non-exclusive, non-transferable, temporary license to use the following software:

- Plasma-SeqSensei™ IVD Software for analysis of NGS data ("SIG Software")

Title, ownership rights and intellectual property rights in the SIG Software shall not pass to the customer. The license is granted in connection with the purchase of Plasma-SeqSensei™ IVD Kits for the duration of the use of the Plasma-SeqSensei™ IVD Kits.

2. Delivery

The SIG Software will be delivered as part of the delivery of the Plasma-SeqSensei™ IVD Kits. The SIG Software will be provided in its current version.

3. Licensed products from third party suppliers

If third party software products are also provided with the SIG Software that are not open-source software, these may only be used in conjunction with the SIG Software. SIG will draw the customer's attention to any special licensing conditions in an appropriate manner.

4. Prohibition of copying

The SIG software as well as the documentation must not be copied by the customer in whole or in part, with the exception of the production of a machine-readable copy of the SIG software for backup or archiving purposes. Any copy made by the customer for these purposes must be clearly and legibly marked with a complete reference to confidentiality, title, ownership rights and the intellectual property rights of SIG.

5. Prohibition of modification

The customer is neither allowed to make any changes to the SIG software himself nor to allow any third party to make any changes.

6. Prohibition of transfer

The transfer of rights and obligations arising from the license agreement to third parties, even after termination of the agreement, is not permitted. It is not permitted to pass on the license key.

7. Unauthorized use

The customer undertakes to ensure that his employees or other persons subject to his instructions who have access to the SIG software comply with all duties of protection and care arising from this agreement. Furthermore, the customer undertakes to ensure that no one gains access to the SIG Software for the purpose of deriving the source codes. If the customer becomes aware that the SIG Software is being used by one of the aforementioned persons in contravention of the existing obligations to protect and exercise due care, he will immediately do everything in his power to prevent such use in contravention of the contract and notify SIG in writing of the use in contravention of the contract.

8. Claim for damages

SIG is entitled to the industrial property rights and copyrights to the SIG Software. The customer can be held liable by SIG for any infringement of such property rights for which he is responsible.

9. Warranty

9.1 For the quality of the SIG Software, only the description of the SIG Software provided by the licensor prior to the conclusion of the contract or agreed in a separate document (e.g. in the documentation) shall be binding. Within the scope of the maintenance obligation, the licensor is not obliged to adapt the software to changed conditions of use and technical and functional developments, such as changes in the IT environment.

9.2 The licensor does not provide any warranty for errors in the software,

- which have been caused by application errors on the part of the customer and which could have been avoided if the documentation had been consulted carefully; this shall also apply in the event of non-existent or insufficient backup measures which would have prevented data loss;
- due to virus attack or other external influences for which the licensor is not responsible, such as fire, accidents, power failure, etc.;
- which are based on the fact that the SIG Software was used in an operating environment other than that approved by the licensor, or which are due to faults in the hardware, the operating system or the software of other manufacturers;
- which are based on the fact that the software has been modified by the customer or third parties without authorisation.

9.3 The customer is obliged to notify the licensor of defects in the SIG Software immediately after their discovery. In the case of material defects, this shall be done by describing the time of occurrence of the defects and the more detailed circumstances. If the licensor carries out a fault analysis at the customer's request and it turns out that there is no defect which the licensor is obliged to rectify, the licensor may invoice the customer for the expenditure incurred on the basis of the licensor's hourly rates valid at the time.

Defects in the software shall be remedied by the licensor within a reasonable period of time (subsequent performance). This shall be done, at the licensor's discretion, by eliminating the defect by means of an update/patch/bugfix/upgrade or by delivering defect-free software or by demonstrating a workaround, the latter to the extent that this is reasonable for the customer, taking into account the effects of the defect and the circumstances of the demonstrated workaround.

10. Liability

10.1 The licensor shall be liable in accordance with the statutory provisions for damages for bodily injury and personal injury, for damages based on the Product Liability Act, for damages caused by fraudulent conduct or intent on the part of the licensor, and for damages caused by gross negligence on the part of the legal representatives or executive employees of the licensor.

10.2 Notwithstanding any liability for damages according to section 10.1, the licensor shall be liable for damages limited to the amount of the foreseeable damage typical for the contract at the time of the conclusion of the contract for damages resulting from a simple negligent breach of essential contractual obligations as well as for damages caused by vicarious agents of the licensor. Material obligations are obligations the fulfilment of which is essential for the proper performance of the contract and compliance with which the licensee may regularly rely on. The contract-typical, foreseeable damage arising from breaches of duty by the licensor shall correspond to the amount of the remuneration paid by the customer in the contract year of the damaging event, up to a maximum of EUR 50,000. If the maximum liability amount is not reached in one contract year, the maximum liability amount for the next contract year shall not be increased.

10.3 Any further liability on the part of the licensor is excluded, subject to any expressly deviating provisions in these General Terms and Conditions. In particular, the licensor shall not be liable for initial defects unless the conditions of Clauses 10.1 or 10.2 are met. The licensor shall not be liable for damages incurred by the Licensee due to failure to back up data.

10.4 Contributory negligence on the part of the customer shall be taken into account.

10.5 The above limitations of liability shall also apply to the personal liability of the licensor's employees, representatives and/or bodies. They also apply to the liability of the licensor with regard to the reimbursement of futile expenses or indemnification obligations.

11. Rights of third parties

If claims are asserted against the customer by third parties due to alleged infringement of a patent, copyright, or other industrial property right to which the third party is entitled in respect of the SIG Software, SIG will indemnify the customer against claims by third parties, provided that the customer informs SIG immediately in writing of the alleged infringement of industrial property rights and supports SIG in the conduct of any legal action.

In the event of such a claim against the customer by a third party, SIG is entitled, at its discretion, either to procure for the customer a corresponding license from the third party, to modify the SIG software or to supply the customer with equivalent other software.

SIG shall not be liable for infringements of property rights resulting from the customer modifying the licensed software or modifying it according to his own requirements, or from the SIG Software being used or sold in conjunction with other software, hardware or consumables not supplied by SIG. This subject matter liability is the entire liability of SIG for infringement of any patent, trademark, copyright, or other intangible property right.

12. Software updates

Updates to the SIG Software will be provided to the customer free of charge.

13. Payment

The license fee is discharged with the purchase of the Plasma-SeqSensei™ IVD Kits. No additional fee will be charged.

14. Contract period

The granted use of the SIG Software shall be valid for the agreed contract period (see clause 1).

The contract may be terminated in writing by either party without notice for good cause. Good cause exists, in particular, if the customer infringes the licensor's rights of use by using the software beyond what is permitted under these General Terms and Conditions and does not remedy the infringement within a reasonable period of time following a warning by the licensor. The licensor reserves the right to assert further claims for damages.

15. Data protection

Insofar as personal data is processed, the licensor shall comply with the statutory provisions on data protection. Details shall be set forth in a Data Processing Agreement to be concluded separately.

16. General provisions

These General Terms and Conditions shall be governed by the laws of Germany. Exclusive place of jurisdiction for all disputes arising from this contract shall be Hamburg.

Should one or more of the provisions of this contract be or become invalid, this shall not affect the validity of the rest of the contract.

No verbal agreements have been made. Amendments and supplements to this contract must be made in writing.

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Systemx Inostics GmbH
Falkenried 88
20251 Hamburg, Germania
www.systemx-inostics.com

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